

Launching Tarheel Reader Books using GoTalk NOW

deannawagner@mac.com

Navigate to TarheelReader.org in the Safari app.

Type your **text** to search.

Use **favorites** if you want, just remember to launch the favorites page if you are using this option.




Option 1:

Scan with one or two switches in GoTalk NOW.

Use auditory cues if needed.

Set the action for each button to launch the URL.

Don't turn on switch control, just use SPACE ENTER interface.

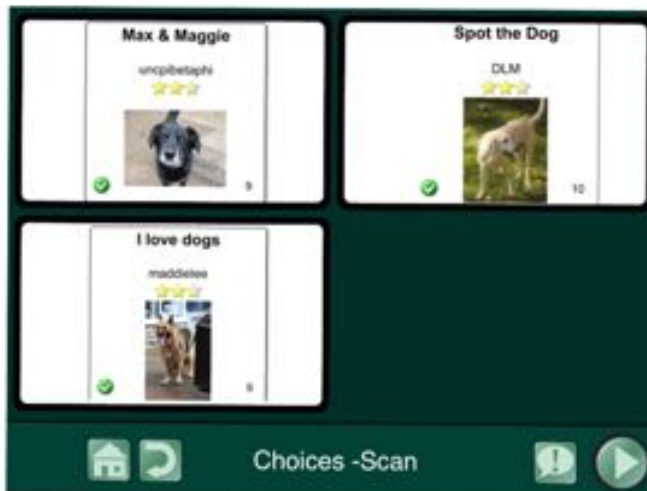

Launch www.tarheelreader.org

Search for your topic.


Open to the first page of your chosen story and take a screenshot.

Next, copy the URL.

Switch to GoTalk NOW and add images to each button, using your screenshot from the photo roll as the button image.

Set the button action to "URL" and paste what you had copied. Repeat for other stories, using a screenshot from the first page and copy/paste of the URL into the GoTalk NOW button.



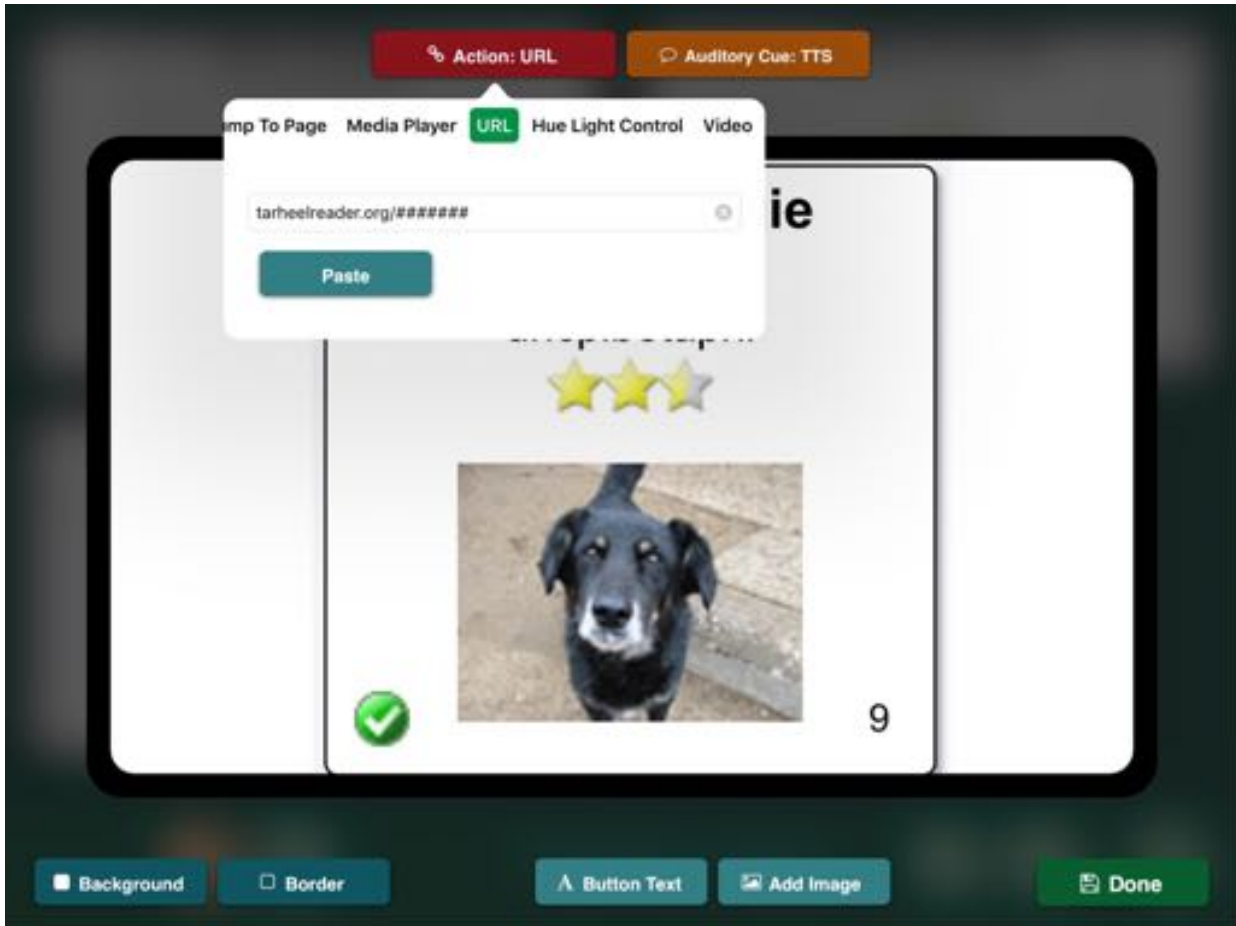


Option #2

Create a custom recipe: switch #1 is right to left swipe and switch #2 is tap the middle of the screen.


Add a third gesture if you have a third switch to swipe right to left (i.e., turn back one page).

Launch your new recipe.

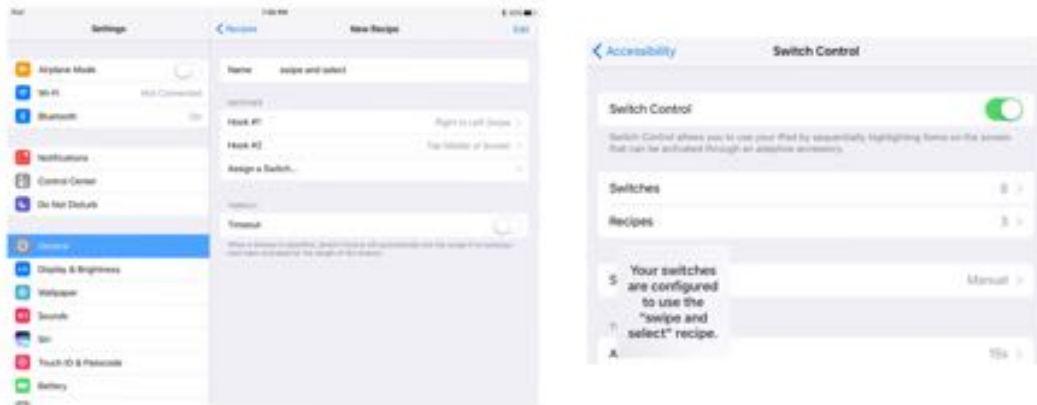

For option #2, using our custom recipe for swipe and select, we will need to make pages with only one button per page.



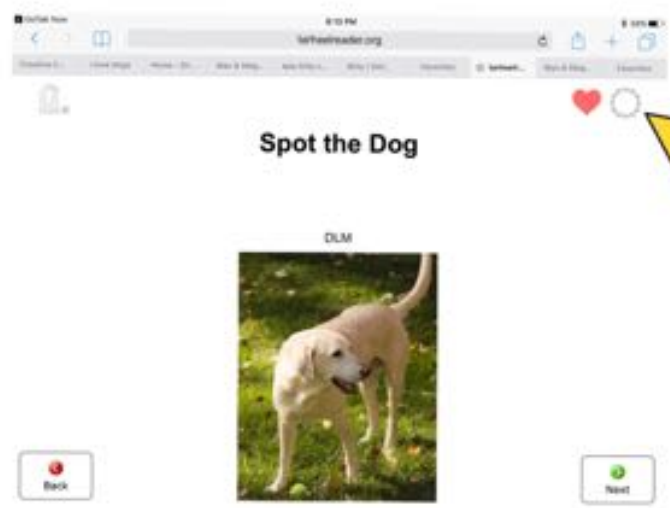

Use settings in TarheelReader to turn on speech if needed.

This will appear on the first page of every story.

After that, you can start reading the story, and switch #1 from your custom recipe will turn the pages.

Touch in the upper left of the screen at any time to return to GoTalk NOW.

